# Versionsinformation Statik-Relax® >> Deutsch

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# Version-Information Statik-Relax® >> English

## ***SR-Display-Version 1.00 (10/2010)***

There is a new alternative solution: The **Statik-Relax-Display**. It has the same functionality like the PDA-program but is much easier to handle because of its fully automated Bluetooth-connection between the Statik-Relax-device and the display.

## ***PC-Version 1.20 (09/2010)***

The "trend-arrow" is now shown regardless of the actual height of the Risk. (But if the energy saving mode is activated the "trend-arrow" is not shown, because the program receives measurement values only every ten minutes.)

A new function for making a "safety-copy" of the Statik-Relax's database is integrated. (Look at the "options"-Button; "medic" and "science"-versions only).

## ***PC-Version 1.10 (02/2010)***

supported operating systems:

- Windows 2000 – service pack 4
- Windows XP (32 & 64 Bit)
- Windows Vista (32 & 64 Bit)
- Windows 7 (32 & 64 Bit)

Values at the time axis on the diagram "risk" (report generation with downloaded data from the Statik-Relax) are now divided in 15, 30 or 60 minutes depending on the duration of the measurement.

An update function is included now.

Every user is allowed to check for available future updates but administrator-privileges (and maybe a valid username and password for the web server) are needed to install the updates. You reach the updater by clicking on the dark blue button at the page "options" in the Statik-Relax-pc-program.

## ***PDA-Version 1.02***

Insertion of a warning message about loose electrodes and EMG signal overmodulation

## ***PDA-Version 1.01***

Adjustment of the control concept on the basis of feedback from the practical use

## ***PC-Version 1.05***

different warning and information messages adapted  
data communication quality during the readout of stored Statik-Relax-data improved

## ***JAVA-Program for mobile phones 1.0***

Launch of JAVA-Program for mobile phones

## ***PDA-Version 1.0***

Launch of PDA-Version (Windows-Mobile®)

## ***PC-Version 1.04***

## **Revision of Bluetooth®-Communication**

The functionality when using BlueTooth®-Stacks not compatible to the Microsoft Windows®-Standard-Stack is improved.

## **Revision of Patient/Proband presentation**

Patients/Probands are now showed in alphabetical order.

## ***Firmware-version 1.3 and PC-version 1.03***

The firmware-version 1.3 requires stringently the PC-software-version 1.03!

## **Change of energy management**

Due to the different characteristics of the accumulator and battery types the automatic device shutdown at a discharged energy store led to very different residual energy quantities remaining in the accumulators or batteries. Especially the use of batteries led to shortened operating times, since special characteristics of the batteries lead to an early automatic shutdown. Because of this reason the energy management of the Statik-Relax was changed. In the current firmware version no automatic device shutdown at a discharged energy store is contained. The device operates as long as the stored energy in the accumulators or batteries is sufficient for powering the device. This leads to a maximization of the operating time for every used accumulator or battery. Approximately 5 to 30 minutes (depending on the type of the accumulator or battery) before the end of the operation time a warning tone sounds twice and from this time the signal lamp flashes yellow in order to signalise the close end of the operation time.

On the biofeedback-window of the PC program starting from the version 1.03 a status indication of the energy store is shown. Choose from the selection box the used type of accumulator or battery to get a correct visualisation of the energy level.

## **Adaption of the volume change sound**

For a better comprehensibility the sound after a button press to change the sound volume was adapted. The Statik-Relax now reacts with a short button acknowledge sound and after that a long sound with the chosen sound volume level (at volume level "off" no long sound).

## **Energy saving mode**

If the energy saving mode is activated by the PC program, only every 10 minutes a risk value of muscular hardening or RSI is requested from the Statik-Relax as before. In the meantime the BlueTooth connection is disconnected. Only the chronological progress of the hardening or RSI risk is shown.

Starting from the version 1.03 the activation of the diagram of the chronological progress of the actual muscular activation switches the energy saving mode off. After that the PC program tries to reconnect to the Statik-Relax. That leads to a short presentation of a lost connection.

Additionally small bugs in the firmware were eliminated.

## ***Firmware-version 1.2 and PC-version 1.02***

Launch of "Medic"-version

## ***Firmware-version 1.1 and PC-version 1.01***

Launch of "Basic"-version